



Figure 1. Parallel Port Location on Furnace Rear Panel

COPY OPERATION PROCEDURE.

Follow the steps below to copy programs between the iBOX and furnace when either the **COPY FURNACE -> iBOX** or **COPY iBOX -> FURNACE** operation is selected.

Important: The iBOX must be connected to and disconnected from the furnace when the corresponding prompts are displayed on the furnace LCD to prevent corruption to programs stored in the iBOX.

1. The furnace LCD will display a **** CONFIRM OPERATION **** message requesting confirmation before the operation is actually performed. Press the **ENTER** key to proceed with the operation or press the **CANCEL** key to abort the operation in case the wrong selection was made.
2. The furnace LCD will display a **CONNECT iBOX** message indicating that the iBOX must now be connected to the furnace parallel port. Connect the iBOX and then press the **ENTER** key to proceed. Alternatively, the **CANCEL** key may be pressed to abort the operation.
3. The furnace LCD will display a **COPYING FURNACE -> iBOX** or a **COPYING iBOX -> FURNACE** message while the corresponding operation is being performed (about 3 seconds) and then will briefly display **COPY OPERATION COMPLETED** message to indicate that the operation was completed successfully.

Note: If a **COPY OPERATION FAILED** message is displayed, then the furnace was not able to copy programs TO or FROM the iBOX.

If one of the following messages is displayed **iBOX IS BLANK** or **iBOX PROGRAMS CORRUPTED** or **iBOX/FURNACE PROGRAM MISMATCH**, then the furnace was not able to copy programs FROM the iBOX.

The mismatch message indicates that the programs stored in the iBOX can not be used for the furnace model that the iBOX is connected to (for example, iBOX is connected to a VIP Universal porcelain furnace while the programs stored in the iBOX are for the VIP Universal X-Press pressable ceramic/porcelain furnace).

4. The furnace LCD will display a **DISCONNECT iBOX** message indicating that the iBOX must now be disconnected from the parallel port. Disconnect the iBOX and then press any key to return the furnace to **IDLE** mode.

VIEW/COPY OPERATION PROCEDURE.

Follow the steps below to view the stored firing programs on the iBOX and then if desired, to copy all programs from the iBOX to the furnace when the **VIEW/COPY iBOX -> FURNACE** operation is selected.

Important: The iBOX must be connected to and disconnected from the furnace when the corresponding prompts are displayed on the furnace LCD to prevent corruption to programs stored in the iBOX.

1. The furnace LCD will display a **CONNECT iBOX** message indicating that the iBOX must now be connected to the furnace parallel port. Connect the iBOX and then press the **ENTER** key to proceed. Alternatively, the **CANCEL** key may be pressed to abort the operation.
2. The furnace LCD will display firing program #0 from the iBOX with a **VIEWING iBOX PROGRAMS** message on the status line. Momentarily press the Up Arrow/0 or Down Arrow/CLEAR keys to scroll up to the next iBOX program or to scroll back to the previous iBOX program, respectively. Press and hold either the **0 (▲)** or **CLEAR (▼)** key to automatically scroll up or down through the iBOX programs, respectively.

Note: If one of the following messages is displayed **iBOX IS BLANK** or **iBOX PROGRAMS CORRUPTED** or **iBOX/FURNACE PROGRAM MISMATCH**, then the furnace was not able to copy programs FROM the iBOX.

The mismatch message indicates that the programs stored in the iBOX can not be used for the furnace model that the iBOX is connected to (for example, iBOX is connected to a VIP Universal porcelain furnace while the programs stored in the iBOX are for the VIP Universal X-Press pressable ceramic/porcelain furnace).

3. When done viewing the iBOX programs, press the **ENTER** key to proceed or press the **CANCEL** key to abort the operation.
4. The furnace LCD will now display **COPY iBOX -> FURNACE ?**. Press the **ENTER** key to copy all programs from the iBOX to the furnace or press the **CANCEL** key to abort this operation.
5. The furnace LCD will display a **COPYING iBOX -> FURNACE** message while the operation is being performed (about 3 seconds) and then will briefly display **COPY OPERATION COMPLETED** message to indicate that the operation was completed successfully.
6. The furnace LCD will display a **DISCONNECT iBOX** message indicating that the iBOX must now be disconnected from the parallel port. Disconnect the iBOX and then press any key to return the furnace to **IDLE** mode.

TECHNICAL DATA.

Connections:	DB25 male connector. Connects to the furnace printer parallel port that supplies operating power and communication signals to the iBOX.
Dimensions:	2.3" W x 2.8" D x 1.0" H
Weight:	1.5 ounces
Operating Temperature:	50 °F to 104 °F
Relative Humidity:	10% to 90%, non-condensing

Air Techniques and ALLPRO Imaging are leading manufacturers of fine dental, medical and veterinary equipment from air and vacuum systems and X-ray film processors, to an impressive line of new products incorporating the most recent technological advances. These new products, vital components of the innovative professional practice, include intraoral cameras, digital imaging systems, which utilizes phosphor plate technology and, most recently, an intraoral digital X-ray system using sensor technology.

Air Techniques and ALLPRO Imaging have been manufacturing quality products for the dental, medical and veterinary professional since 1962.

Air Techniques and ALLPRO Imaging products are distributed only through authorized dealers. Refer to www.airtechniques.com or www.allproimaging.com to find a dealer in your area.

- Accent™ Intraoral Digital X-ray Image System
- Acclaim® Intraoral Digital Video Camera System
- Acclaim® USB Only Intraoral Digital Video Camera System
- AirStar®
- A/T 2000® XR
- Guardian™ Amalgam Collector
- Peri-Pro®
- Provecta 70™
- Rinsendo Root Canal Disinfection System
- ScanX®
- STS™
- VacStar™

**AIR
TECHNIQUES
INC.**
1-800-AIR-TECH
(1-800-247-8324)
www.airtechniques.com

- 100 Plus
- 2010
- Medscope
- Provecta V
- ScanX® 12
- ScanX® DVM
- ScanX® 12 DVM
- ScanX® NDT
- ScanX® 12 Portable
- ScanX® 14 Portable
- ScanX® NDT Portable
- ScanX® 14 In-Counter



ALLPRO IMAGING
1-800-AIR-TECH
(1-800-247-8324)
www.allproimaging.com